

BEFORE USE PLEASE READ THE FOLLOWING INSTRUCTIONS CAREFULLY
PRODUCT FOR DENTAL USE ONLY

GLUCO-CHEX GEL 2%

Gel for root canals preparation

COMPOSITION: chlorhexidine digluconate, additional substances

INTENDED USE

Product *GLUCO-CHEX GEL 2%* is intended for use during endodontic treatment, it simplifies mechanical root canal preparation. Product moisturizes root canal during its mechanical preparation, lubricants contained in the product enable easy introduction of the instrument into the canal and the risk of its breakage is minimized.

INSTRUCTIONS FOR USE

Introduce the product into the root canal directly on the canal instrument or by means of attached applicator, at the same time prepare the canal mechanically. While preparing the canal rinse it with physiological saline or distilled water.

CAUTION! Do not rinse with sodium hypochlorite since it may cause precipitation of a tawny deposit which is hard to remove.

Product intended for multiple use except the applicator which is for single use only.

Multiple use of applicator may cause the risk of reinfection.

CONTRAINDICATIONS

Do not use the product for patients with hypersensitivity to the product's components.

Do not use directly with sodium hypochlorite because of the risk of precipitation of a tawny deposit which is hard to remove.

CAUTIONS

Caution should be taken during canal preparation close to the apex area, because it may cause perforation of the canal as a result of use of root canal instruments.

It may cause irritation to the mucous membrane of the oral cavity and eyes.

In case of contact rinse immediately with copious amount of water.

In case of consumption do not extract emesis. Drink copious amount of water. Seek medical advice.

STORAGE

Store in original packaging at temperature below 25 °C.

Store out of reach of children.

Expiry date is placed on the main container.

Once container is open, the expiry date doesn't change if it is securely closed after each use.

HANDLING THE PRODUCT CONTAINER

Empty out containers should be disposed of responsibly.

PACKAGING

Syringe of 5ml or 10ml, dispenser, connector, disposal applicators .



www.medicando.gr

info@medicando.gr

Tel: 210 2582902 Fax: 210 2582905


1011

medicando



www.medicando.gr
info@medicando.gr

Tel: 210 2582902 Fax: 210 2582905